
# VitalStim® THERAPY

## Instructions for Application and Removal of Electrodes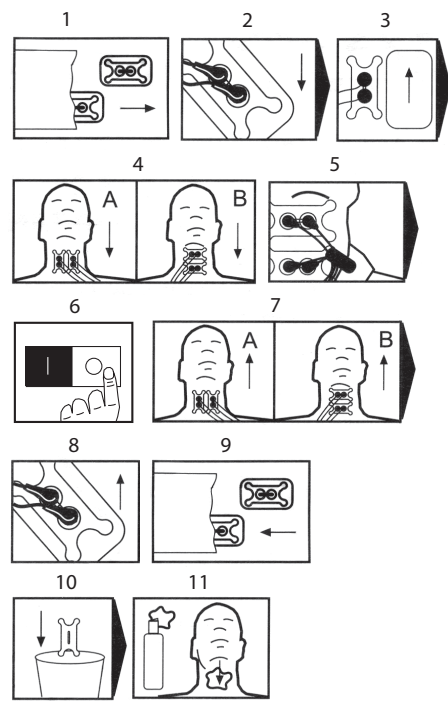

Manufactured in China for:  
**DJO, LLC**  
1430 Decision Street  
Vista, CA 92081 U.S.A.  
**USA 800.506.1130**  
Outside USA +33 (0)5 59 52 86 90

EC	REP
----	-----

**EUROBYTECH**  
88 rue de la Liberté  
71000 MACON  
FRANCE

**GMDASZ**  
Manufacturing Co., Ltd  
Shenzhen P.R.C  
Made in China

**chattanooga**  
13-59047 Rev A © 2017 DJO, LLC.

ENGLISH

# VitalStim® THERAPY

## Electrode Application Kits. For Use with VitalStim® Therapy Only.

This equipment is to be used only by a licensed practitioner certified in the use of the VitalStim® Therapy.

### Single Patient Use Electrode Application Kits

#### Application Instructions:

1. Remove VitalStim electrodes from pouch.
2. Snap lead wires onto stems of VitalStim electrode.
3. Remove VitalStim electrode from the liner and retain for storage.
4. Apply VitalStim electrode to appropriate skin area for application A or B or as instructed at your VitalStim® Therapy certification program. Note: Discard the VitalStim electrode if it does not adequately adhere to the patient's skin for the second session.
5. Attach lead wire strain relief clips to the patient's clothing in a position that allows ease of movement and adequate strain relief for the lead wires.

#### Removal instructions:

6. Turn off VitalStim stimulation unit.
7. Lift one edge of the VitalStim electrode and peel. DO NOT pull on the lead wires. Doing so may irritate skin and damage lead wires.
8. Remove VitalStim electrode from lead wires.
9. After first use (if a second use is desired): return electrode to the liner and roll down the top of the pouch for storage. When the patient is ready to begin the second session proceed to #1 of the Application Instructions section. Note: If the electrode does not fully adhere discard the electrode. Caution: This device is intended to be used on a single patient only. With proper care, the electrodes may be reused a 2nd time before discarding.
10. Discard VitalStim electrode.
11. Apply aloe vera gel as needed to skin at stimulation sites.

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